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SCHWEBETISCH P- EURO

User manual Suspension Table P-EURO

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TABLE OF CONTENTS	Page:
1. Operating elements	3
2. Loading and unloading of stretcher	7
3. Lifting - lowering	10
4. Head-end lowered/foot-end lowered/suspension/lock	12
5. Change between moving stretcher/ DIN- stretcher	13
6. Cross- sliding mechanism	14
7. Emergency (lowering) draining	15
8. Safety hints/ Preventing accidents	16
9. Technical Data	17
10. Manufacturer Details	18
11. Declaration of conformity	19

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
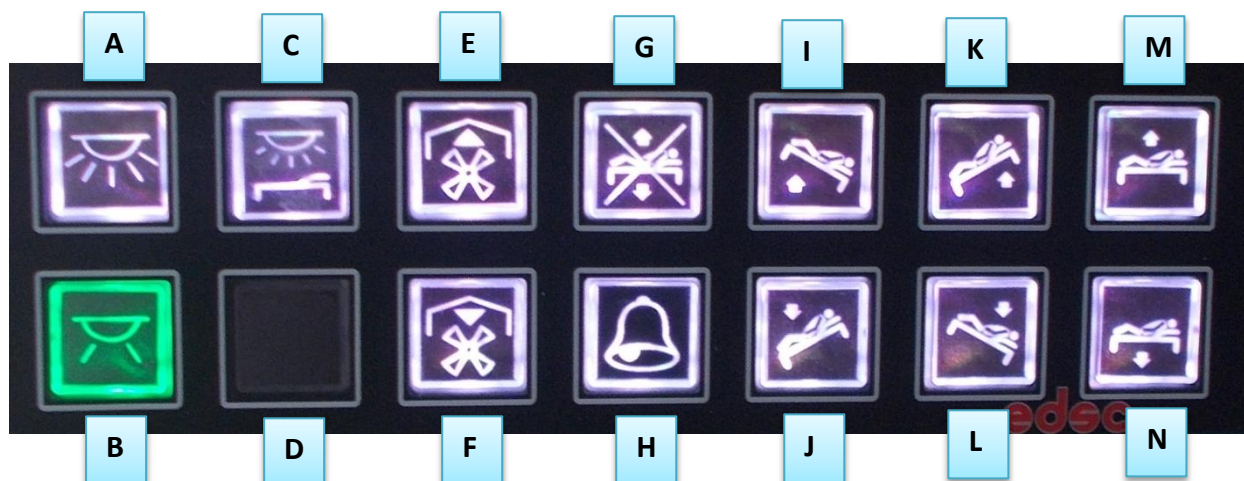
The suspension table will be operated from switchboard in patient's compartment. Lettering of the switches is done acc. To their functions. Each control lamp is showing the block position as well as the switched "down-position".

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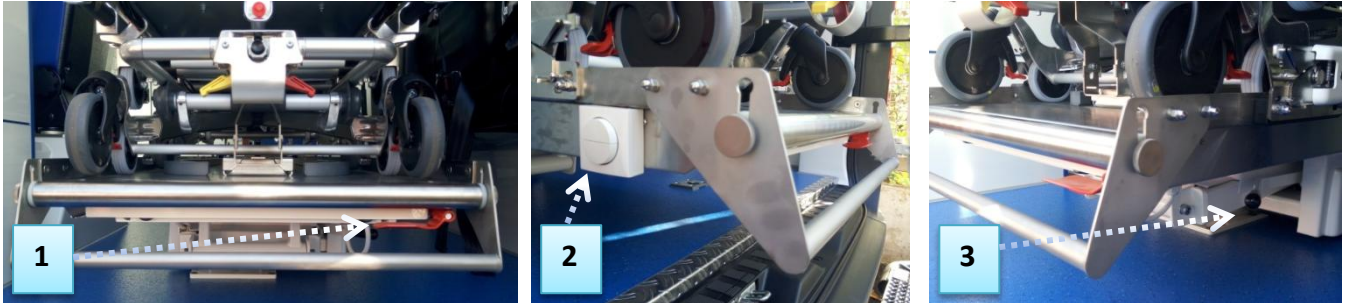Following operating elements are placed on the back side of the suspension table:

1. unlocking upper platform
2. switch lifting-lowering end of platform
3. unlocking of cross-sliding mechanism

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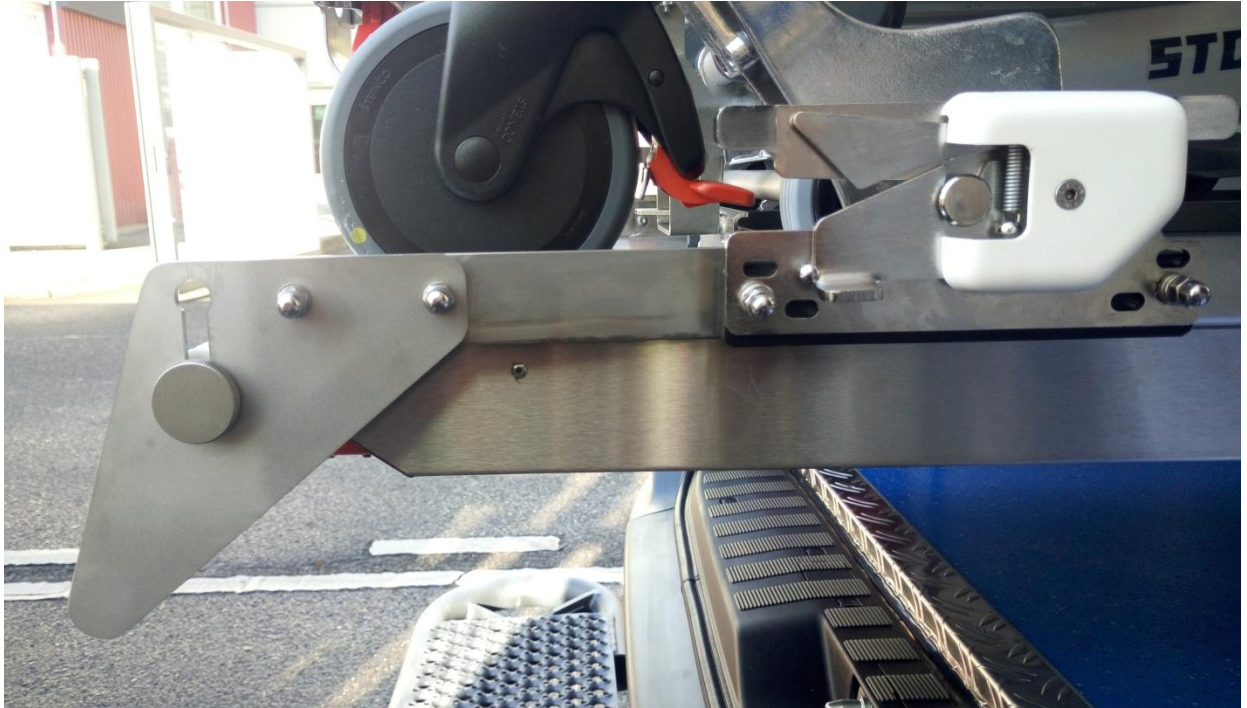
The foot bracket for unlocking of the cross-sliding mechanism is positioned on front side.

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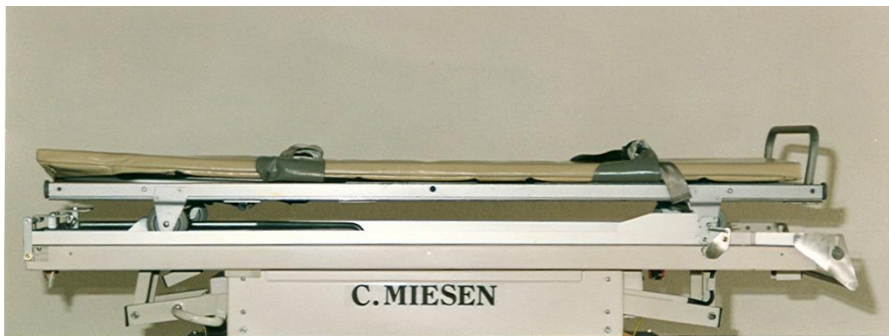
An adjustable roller as roll-down security for DIN-stretcher as well as pull-up roll for moving stretcher is positioned at the rear end of upper platform. The locking device for DIN- stretcher is positioned on rear LH side.

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Loading and unloading of stretcher

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For loading and unloading, the platform must be positioned in a horizontal position by using the resp. "center"- switch. For lowering platform you can choose between using the "down"-switch or pull out platform. In last case the platform is lowering automatically.

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Before pulling out the upper platform, the platform must be unlocked by pressing handle 1. After pulling-out the upper platform, switch 3 must be pressed until the end of the platform is lowered into requested height. The lowering of the end of platform can be stopped when not pressing switch 3. Loading and unloading of stretcher in lowered position of the end of platform. When loading stretcher pay attention to safety locking of stretcher.

Attention: While unlocking the stretcher it's imperative to stand behind it, don't let the stretcher roll out uncontrolled. Follow the Operating manual of the stretcher.
Take care of Infusion Tubes and ECG cables while loading or unloading the stretcher, Danger of squeezing ore straining the lines.

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9 USER MANUAL

For lifting the end of platform, pull switch 3 so long, until platform reaches horizontal position. The lifting of the end of platform can be interrupted by not pressing switch 3. Pull in platform, after it will be unlocked by pressing handle 1. Pay attention to safety locking of platform.

Attention: The Patient has to be adequately secured to the stretcher, take care that no part of the patients body reaches over the limits of the stretcher, danger of crushing.
Take care that there are no other People in the bending range of the Suspension table.

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For lifting and lowering the platform press "up " resp. "down"-switch. The up and down can be interrupted at each time by not pressing the switch. Tip: If the platform shall be lowered for loading resp. Unloading of the stretcher, the "down"-switch must be pressed so long, until "down" lamp goes off.

Attention: The Patient has to be adequately secured to the stretcher, take care that no part of the patients body reaches over the limits of the stretcher, danger of crushing. Take care that there are no other People in the bending range of the Suspension Table.

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4. KOPFTIEF-FUSSTIEF, FEDER-BLOCK

Head-end lowered; foot-end lowered, Suspension-lock
Proclive-déclive, suspension blocage

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To bring the platform into head-end resp. foot-end lowered position, the corresponding switch must be used. The head-end resp. foot-end lowered position can start from each position of the platform. The horizontal position can be reached by pressing resp. "center"-switch. This switch must be pressed so long, until platform reaches the requested position. A locking of the suspension can be reached by pressing switch "lock". When suspension is locked, the "lock-control-lamp" will illuminate.

Attention: The Patient has to be adequately secured to the stretcher, take care that no part of the patients body reaches over the limits of the stretcher, danger of crushing. Take care that there are no other People in the bending range of the Suspension Table.

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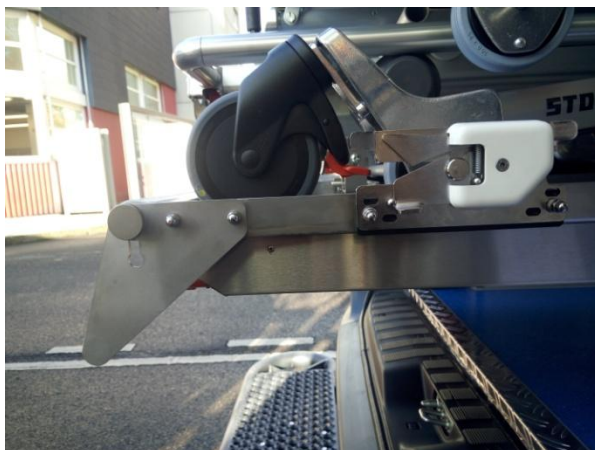
Change between moving stretcher and DIN- stretcher

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The rear roller must be in upper position for transport of a DIN- stretcher, to prevent the unrolling of the stretcher after unlocking. For transport of a moving stretcher the roller has to be changed to lower position for loading resp. unloading. The change between lower and upper position takes place by pressing the right and left knob and simultaneously lifting resp. lowering of the roller. Please note, that the roller safety locks in upper position.

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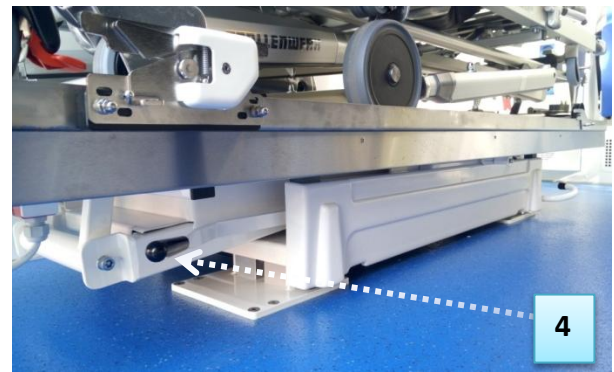
For cross-sliding of the complete Treatment Table the front foot step resp. rear handle must be operated. Pay attention to safety lock of Treatment Table in each position.

Attention: The Patient has to be adequately secured to the stretcher, take care that no part of the patients body reaches over the limits of the stretcher, danger of crushing. Take care that there are no other People in the bending range of the Suspension Table. Don't use the Cross-sliding mechanism while cornering.

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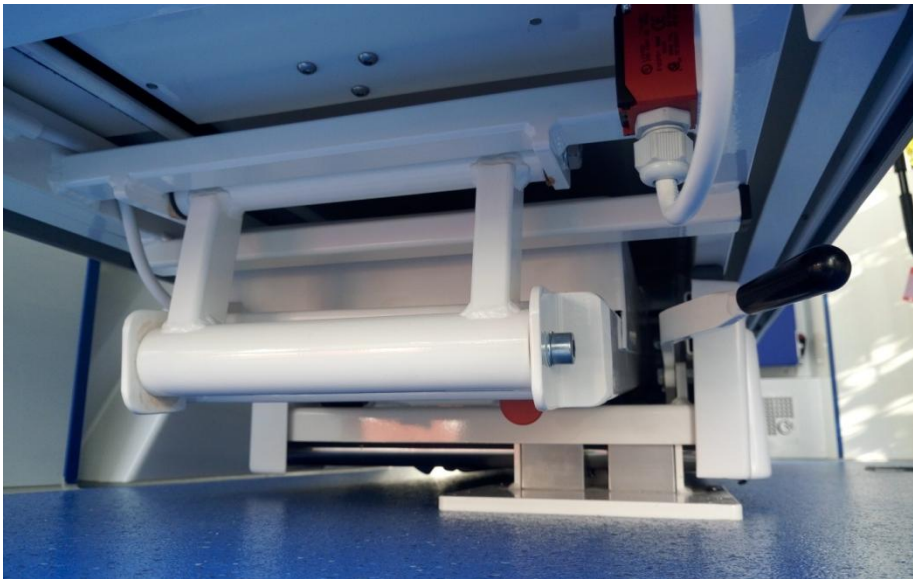
On rear end of treatment table you find the draining screw knob for emergency lowering. By failure in hydraulic resp. electric system turn left the knob smoothly and let go immediately after platform is lowering.

Attention:

Do not put hand or arm in lever- mechanic. Danger of squeezing.

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We declare under sole responsibility that the product described above is in compliance with directive 93/42/EEC. The product is CE marked.

Bonn, 17. Mai 2013

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© Copyright **C.MIESEN GmbH & Co. KG**

The published images, photos, graphics, illustrations, text and layout of this press report are protected by international copyright law. Any unauthorized use, reproduction or distribution of this press report or even individual content is without the written permission of C.MIESEN GmbH & Co. KG is not allowed.

Date of print: Oktober 2015

www.miesen.de

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C. MIESEN GmbH & Co. KG